SiltraX® AS Braided Gingival Retraction Cord with Aluminum Sulfate

Siltra X AS is braided cotton cord saturated with a solution of aluminum sulfate and dried to a technique which retains the activity of the active ingredient. Normal tissue moisture reactivates the active ingredient, which promptly retracts gingival tissue in a manner that causes no tissue damage, with full recovery upon drying.

Contents: Each bottle contains 183cm braided cotton cord. The average concentration of aluminum sulfate for each size is:
 Siltra X 7/00 0.12 mg/ml
 Siltra X AS 0 0.189 mg/ml
 Siltra X AS 9/1 0.335 mg/ml
 Siltra X AS 10/2 0.57 mg/ml

Indications for use: Use whenever gingival retraction is advisable in dental practice, as precontaining impression taking with any elastic impression material, for or to placement of caps, bridges, and denture partial appliances.

Precautions: There are no known contraindications for the use of Siltra X AS. The chemical components of Siltra X AS have a long history of safety and effectiveness for the purposes intended.

Warnings: For topical intraoral use only. Do not ingest. Keep out of reach of children.

- Read and follow all instructions.
- For professional in-office use only.
- For topical intraoral use only. Do not ingest.
- Keep out of reach of children.
- Do not use with patients with aluminum sulfate allergy or sensitivity.

Amount to use: Use as much cord of the appropriate size as is necessary to pull the way around the neck of each retraction with a slight overlap. Two loops may be placed around each tooth (see equivalent), if sulcus space allows. Retraction with an epithemeric cord should be limited to the equivalent of one normal quadrant of the mouth per appointment.

Directions for Use:

- For most effective use of Siltra X cord, strand a strand of material around the tooth and over the crown.
- Trim the ends of string overlap, and pack the loop firmly into sulcus.
- Apply a second loop of larger cord over the smaller cord in the same manner.
- When ready to apply the impression material, remove the larger cord loop only, leaving the smaller cord in place below the gingival margin until the impression procedure has been completed.
- Once the impression procedure has been completed the smaller cord is removed also.
- Alternately, a single loop of cord will suffice.
- Siltra X AS completely retracts tissue within five to eight minutes and should then be removed from the mouth.

Disposed strands of cord are intended for single use only and should be disposed of immediately after use. Reusing spent strands puts patients at increased risk of injury or infection.

For product SDS information please see company website at pasdent.com or contact Pasdent at 407 329 1234.

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