Retrax® Gel AS / Retrax® Gel AS Clear
Each 0,75 gram single-use syringe contains 25% Aluminum Sulfate Gel

Indications for Use:

Retrax® Gel AS/Retrax® Gel AS Clear is intended to be used with all types of retraction cord to aid in gingival retraction and hemostasis. Retrax® Gel AS/Retrax® Gel AS Clear makes initial cord packing easier by providing lubrication when packing cord, allowing the cord to glide into the sulcus. Retrax® Gel AS/Retrax® Gel AS Clear may also be used during any retraction where gingival retraction and hemostasis are indicated. This product is for topical in-office use only. Each cartridge is intended for single use. Retrax® Gel AS/Retrax® Gel AS Clear is compatible with all of Pascal's medicated and plain cords.

Contraindications: None known.

Warnings: For topical intraoral use only. Do not ingest. Keep out of reach of children.

Precautions:

- For professional office use only.
- Do not use on patients with an allergy to aluminum sulfate.
- Retrax® Gel AS contains artificial color and flavor; confirm that no known patient allergies exist.
- Retrax® Gel AS Clear contains no artificial color, but does contain artificial flavor.
- High volume suction is recommended to avoid possible ingestion.
- Reusing Syringe puts patients at increased risk for injury or infection.

Dosage: Use as needed to achieve and maintain hemostasis. Under normal usage, each single-use syringe should treat up to two application sites.

Directions for Use:

- The 0,75 gram pre-filled syringe should be used together with a disposable applicator tip or MicroFuser™ tip.
- Remove the syringe cap and attach the tip.
- For best results, withdraw syringe plunger slightly prior to initial use.
- Gently apply a thin bead of gel directly to the sulcus area.
- Select cord and pack into gel. Wait 2 - 6 minutes.
- If bleeding persists, rinse area and reapply gel. Wait 2 - 6 minutes.
- Remove cord.
- For best results, thoroughly rinse and dry application site.
- Take Impression as usual.
- Discard syringe and unused Retrax® Gel AS.

The syringes are recommended for single use only and should not be reused or autoclaved.

For product SDS information please see company website at pascaldental.com or contact Pascal directly.

Pascal's Retrax® Gel AS & Retrax® Gel AS Clear are free of gluten, soy, milk, egg, peanut and treenut products.

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