EN

Retrax® Gel FS

Each 0.75 gram single-use syringe contains 15.5% Ferric Sulfate Gel

Indications for Use:

For the purpose of aiding in gingival retraction and to suppress excessive capillary bleeding associated with dental impression procedures, as well as for tissue management problems in other phases of dentistry. This product is for topical use only and each syringe is intended for single use. Retrax® Gel FS is compatible with all types of retraction materials except those containing epinephrine or aluminum chloride.

Contraindications: None known.

Warnings: For topical intraoral use only. Do not ingest. Keep out of reach of children.

Precautions:

- For professional office use only.
- Do not use on patients with an allergy to ferric sulfate.
- Do not use Retrax® Gel FS with products containing epinephrine.
- Temporary tissue discoloration may occur. This discoloration should last no more than 24 – 48 hours.
- Select cord and pack into gel. Wait 1 – 3 minutes.
- High volume suction is recommended to avoid possible ingestion.
- Reusing Syringe puts patients at increased risk for injury or infection.

Dosage: Use as needed to achieve and maintain hemostasis. Under normal usage, each single-use syringe should treat up to two application sites.

Directions for Use:

- The 0.75 gram pre-filled syringe should be used together with a disposable applicator tip or MicroFuser™ tip.
- Remove the syringe cap and attach the applicator tip.
- For best results, withdraw syringe plunger slightly prior to initial use.
- Gently apply a thin bead of gel directly to the sulcus area.
- Select cord and pack into gel. Wait 1 – 3 minutes.
- If bleeding persists, rise area and reapply gel. Wait 1 – 3 minutes.
- Remove cord.
- For best results, thoroughly rinse and dry application site.
- Take impression as usual.
- Discard syringe and unused Retrax® Gel FS.

The syringes and tips are recommended for single use only and should not be autoclaved.

For product SDS information please see company website at pascaldental.com or contact Pascal Directly.

Pascal's Retrax® Gel FS is free of gluten, soy, milk, egg, peanut and treenut products. Retrax® Gel FS contains no artificial color or flavor.

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