

Retrax® Gel AS

Indications for Use:

Retrax Gel AS is intended to be used with all types of retraction cord to aid in gingival retraction and hemostasis. Retrax Gel AS assists in cord packing by providing lubrication, allowing the cord to glide into the sulcus.

Retrax Gel AS may also be used during any restoration where gingival retraction and hemostasis are indicated.

This product is for topical in-office use only. Retrax Gel AS is compatible with all of Pascal's medicated and plain cords.

Contraindications: None known.

Warnings: For topical intraoral use only. Do not ingest. Keep out of reach of children.

Precautions:

- For professional office use only.
- Do not use on patients with an allergy to aluminum sulfate.
- Retrax Gel AS contains artificial color and flavor; confirm that no known patient allergies exist.
- High volume suction is recommended to avoid possible ingestion.
- The contents of this jar should be used within 90 days of opening.

Dosage: Use a suitable applicator to apply an amount sufficient to spread a thin film or bead around the sulcus. Apply additional amounts as needed.

Directions for Use:

- Remove and discard the foil seal on jar. The foam liner should stay inside the cap to help seal jar and preserve gel when not in use.
- Retrax Gel AS should be stirred prior to each use.
- Gently apply gel directly to the sulcus area.
- Select cord and pack into gel.
- Alternately, a small amount of gel can be dispensed and retraction cord can be dipped into gel prior to packing in sulcus.
- Wait 2 - 6 minutes.
- If bleeding persists, rinse area and reapply gel. Wait 2 - 6 minutes.
- Remove cord.
- For best results, thoroughly rinse and dry application site.
- Take impression as usual.

For product SDS information please see company website at pascaldental.com or contact Pascal directly.

Pascal's Retrax Gel AS is free of gluten, soy, milk, egg, peanut and tree nut products.

SN70075/1219

CE  86°F (30°C)

 **Pascal**®

 2929 Northup Way
Bellevue, WA 98004 USA
425.827.4694 • 800.426.8051
www.pascaldental.com

 EC REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands