REF 16-600 RETRAX® SOLUTION 15mL

EN

Indications for Use:

Retrax Solution is intended to be used with all types of retraction cord and cotton pellets to aid in gingival retraction and hemostasis. Retrax Solution makes initial cord packing easier by providing lubrication within packing cord, allowing the cord to glide into the sulcus. Retrax Solution may also be used during any restoration where gingival retraction and hemostasis are indicated. This product is for topical in-office use only. Retrax Solution is compatible with all of Pascal's medicated and plain cords and pellets.

Contraindications: None known.

Warnings: For topical intraoral use only. Do not ingest. Keep out of reach of children.

Precautions:

- For professional office use only.
- Do not use on patients with an allergy to aluminum sulfate.
- Retrax Solution contains artificial flavor; confirm that no known patient allergies exist.
- High volume suction is recommended to avoid possible ingestion.

Dosage: Use as needed to achieve and maintain tissue retraction and hemostasis.

Directions for Use:

- Soak cotton pellet or retraction cord in dispensed solution.
- Apply pellet or cord to gingival site.
- Wait 2 - 6 minutes.
- If bleeding persists, rinse area and reapply soaked pellet or cord.
- Wait 2 - 6 minutes.
- Remove cord or pellet.
- For best results, thoroughly rinse and dry application site.
- Take impression as usual.

Retrax Solution should only be applied using cotton pellets, retraction cord, or cotton tipped applicators. Solution should never be dispensed directly into patient's mouth or directly to tissue.

For product MSDS information please see company website at pasccidental.com or contact Pascal directly.

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