

## Retrax® Gel FS

### Indications for Use:

For the purpose of aiding in gingival retraction and to suppress excessive capillary bleeding associated with dental impression procedures, as well as for tissue management problems in other phases of dentistry. This product is for topical use only and each syringe is intended for single use. Retrax Gel FS is compatible with all types of retraction materials except those containing epinephrine or aluminum chloride.

**Contraindications:** None known.

**Warnings:** For topical intraoral use only. Do not ingest. Keep out of reach of children.

### Precautions:

- For professional office use only.
- Do not use on patients with an allergy to ferric sulfate.
- High volume suction is recommended to avoid possible ingestion.
- Temporary tissue discoloration may occur. This discoloration should last no more than 24 – 48 hours.
- Do not use Retrax Gel FS with products containing epinephrine.
- The contents of this jar should be used within 90 days of opening.

**Dosage:** Use as needed to achieve and maintain hemostasis.

### Directions for Use:

- Remove and discard the foil seal on jar. The foam liner should stay inside the cap to help seal jar and preserve gel when not in use.
- Retrax Gel FS should be stirred prior to each use.
- Gently apply gel directly to the sulcus area.
- Select cord and pack into gel.
- Alternately, a small amount of gel can be dispensed and retraction cord can be dipped into gel prior to packing in sulcus.
- Wait 1 - 3 minutes.
- If bleeding persists, rinse area and reapply gel. Wait 1 - 3 minutes.
- Remove cord.
- For best results, thoroughly rinse and dry application site.
- Take impression as usual.

For product SDS information please see company website at [pascaldental.com](http://pascaldental.com) or contact Pascal directly.

Pascal's Retrax Gel FS is free of gluten, soy, milk, egg, peanut and tree nut products. Retrax Gel FS contains no artificial color or flavor.

SN70765/1219



 **Pascal**



2929 Northup Way  
Bellevue, WA 98004 USA  
425.827.4694 • 800.426.8051  
[www.pascaldental.com](http://www.pascaldental.com)



EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands