

# Pascord®

## Twisted Gingival Retraction Cord with Aluminum Sulfate

Pascord is twisted cotton cord saturated with a solution of aluminum sulfate and dried by a technique which retains the activity of the medication. Normal tissue moisture reactivates the medication, which promptly retracts gingival tissue in a manner that causes no tissue damage, yet fully exposes the gingival margin.

**Contents:** Each bottle contains 72 inches twisted cotton cord. The average concentration of aluminum sulfate for each size is:

Pascord 7/00	0.30 mg/in
Pascord 8/0	0.48 mg/in
Pascord 9/1	0.85 mg/in
Pascord 10/2	1.45 mg/in

**Indications for Use:** Use whenever gingival retraction is advisable in dental practice, as preceding impression taking with any elastic impression material; or prior to placement of caps, crowns, bridges; and during tooth preparation.

**Contraindications:** There are no known contraindications for the use of Pascord. The chemical components of Pascord have a long history of safety and effectiveness for the purposes intended.

**Warnings:** For topical intraoral use only. Do not ingest. Keep out of reach of children.

**Precautions:**

- Read and follow all instructions.
- For professional in-office use only.
- Do not use on patients with aluminum sulfate allergy or sensitivity.

**Amount to Use:** Use as much cord of the appropriate sizes as is necessary to go all the way around the neck of each tooth to be retracted, with a slight overlap. Two loops may be packed around each tooth (see below), if sulcus space allows.

**Directions for Use:**

- For most effective use of Pascord, loop a strand of smaller cord around the tooth and over the crevice.
- Trim the ends for a slight overlap, and pack the loop firmly into the sulcus.
- Apply a second loop of larger cord over the smaller cord in the same manner.
- When ready to apply the impression material, remove the larger cord loop only, leaving the smaller cord in place below the gingival margin until the impression procedure has been completed.

- Once the impression procedure has been completed the smaller cord is removed also.
- Alternately, a single loop of cord will suffice.
- Pascord completely retracts tissue within five to eight minutes and should then be removed from the mouth.

Dispensed portions of cord are intended for single use only and should be disposed of immediately after use.

For product MSDS information please see company website at [pascaldental.com](http://pascaldental.com) or contact Pascal directly.

Pascal's Pascord is free of gluten, soy, milk, egg, peanut, and tree nut products. Pascord contains no artificial flavors.

Brochures describing our latest gingival retraction materials are available upon request, or visit us at [pascaldental.com](http://pascaldental.com).



**2929 NE Northup Way  
Bellevue, WA 98004 USA  
425.827.4694 800.426.8051  
[www.pascaldental.com](http://www.pascaldental.com)**

**OPEN HERE**

RX ONLY (in USA)  
For professional use only. Read package insert for additional cautions and warnings. This product is not sterile. Single use only. Dispensed portions not to be reused.



**2929 NE Northup Way  
Bellevue, WA 98004 USA**

ISO 9001 (QMS)



ISO 13485 (QMS)



0086

[www.pascaldental.com](http://www.pascaldental.com)