**Retrax® Gel AS / Retrax® Gel AS Clear**  
Each 0.75 gram single-use syringe contains 25% Aluminum Sulfate Gel

#### Indications for Use:

Retrax® Gel AS/Retrax® Gel AS Clear is intended to be used with all types of retraction cord to aid in gingival retraction and hemostasis. Retrax® Gel AS/Retrax® Gel AS Clear makes initial cord packing easier by providing lubrication when packing cord, allowing the cord to glide into the sulcus.  
Retrax® Gel AS/Retrax® Gel AS Clear may also be used during any retraction where gingival retraction and hemostasis are indicated. This product is for topical in-office use only. Each cartridge is intended for single use. Retrax® Gel AS/Retrax® Gel AS Clear is compatible with all of Pascal's medicated and plain cords.

**Contraindications:** None known.

**Warnings:** For topical intraoral use only. Do not ingest. Keep out of reach of children.

#### Precautions:

- For professional office use only.
- Do not use on patients with an allergy to aluminum sulfate.
- Retrax® Gel AS contains artificial color and flavor; confirm that no known patient allergies exist.
- Retrax® Gel AS Clear contains no artificial color, but does contain artificial flavor.
- High volume suction is recommended to avoid possible ingestion.
- Reusing Syringe puts patients at increased risk for injury or infection.

**Dosage:** Use as needed to achieve and maintain hemostasis. Under normal usage, each single-use syringe should treat up to two application sites.

#### Directions for Use:

- The 0.75 gram pre-filled syringe should be used together with a disposable applicator tip or MicroFuser™ tip.
- Remove the syringe cap and attach the tip.
- For best results, withdraw syringe plunger slightly prior to initial use.
- Gently apply a thin bead of gel directly to the sulcus area.
- Select cord and pack into gel. Wait 2 - 6 minutes.
- If bleeding persists, rinse area and reapply gel. Wait 2 - 6 minutes.
- Remove cord.
- For best results, thoroughly rinse and dry application site.
- Take Impression as usual.
- Discard syringe and unused Retrax® Gel AS.

The syringes are recommended for single use only and should not be reused or autoclaved.

For product SDS information please see company website at [pascaldental.com](http://pascaldental.com) or contact Pascal directly.

Pascal's Retrax® Gel AS & Retrax® Gel AS Clear are free of gluten, soy, milk, egg, peanut and tree nut products.

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